

Procedure to Calibrate Pressure Sensors For MARK V Controls

Make sure that the machine has NO PRESSURE IN JACKET OR CHAMBER. The best time to perform this calibration is first thing in the morning, before the jacket has been warmed up. You may also do this procedure at other times, but you must first DEPRESSURIZE THE JACKET, which might take a while.

Once the JACKET PRESSURE has been reduced to ZERO, make sure that the STERILIZER DOOR IS OPEN. From the Main Menu, press 5 if there are three entries on the main menu (press 4 if there are only 2 entries). When prompted for a CE CODE, press 2727 and then press ENTER. From the ensuing menu, select

5) CALIBRATE PRESSURE TRANSDUCERS.

From there, you will be instructed to make sure that BOTH THE CHAMBER AND JACKET ARE AT ATMOSPHERE, and when they are to hit the RUN key. At this time the JACKET AND CHAMBER PRESSURE READINGS ON THE CRT WILL SHOW ZERO. Press 9 to go all the way back to the MAIN MENU. Press JACKET button and pressurize the JACKET. You are ready to go.